SELECT PROFCARE

PROFCARE D

Style no. 720120 / 720121

Material:

Fabric: 75% Nylon, 25% Elastane.
Glue: 100% Acrylic adhesive

Medical Device Class I, Rule 1, for non-sterile and noninvasive devices for short term use, by the European Medical Device Regulative 2017/745.

Basic UDI-DI: 5703543U1022LA

Made in China

For further information and inquiries regarding this product, please contact SELECT SPORT A/S

Version: 01.01.2025

SELECT SPORT A/S
Fabriksparken 46
DK-2600 Glostrup

+45 43 96 96 66 - select@select.dk

SELECT-SPORT.COM

EN

Intended use:

The Profcare D is a new type of highly elastic sportstape with references to the kinesiology area. With multiple stretch directions and strong water repellent fabrics this tape will work perfectly in symbiosis with big movements – making it a great tool for injury rehabilitation, support, or motor control. Due to its high mobility, heavy fabric, and straight-lined mold, it will impact sensory feedback and provide better conditions for blood flow and movement. See it as a hybrid between ordinary rigid sportstape and an elastic wrap.

Instructions for use:

Tear of the grid paper and place the base-slip on skin with no stretch- as a base. Pull and stretch the tape in a direction and amount you feel is appropriate, place it on the skin, place your second base with no stretch and heat up the tape with grid paper friction to increase durability. It's recommended to use an adhesive spray before to optimize attachment in wet conditions. Duration of use is maximum 3-5 days.

Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients.

Contraindications:

Use of the product is contraindicated when:

- The skin is broken. Do not stick on a wound directly.
- There is significantly swelling at the application site.
- The user has known allergies to nylon, glue, or Elastane.
- The user is known to have a vulnerable or very dry skin.

Warnings:

Do not apply too tight. Ensure that the skin is cleaned before application. Do not store above 25 degrees Celsius. Store in original packaging to protect from sun light. Symptoms of overexposure of the acrylic acid may include redness or discoloration, swelling, itching, burning, or blistering. Consult your Health Care Provider if you experience any reaction.

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